



Product information

Prophylactic Powder

TOP Lemon	SUN435Q
TOP Mint	SUN436Q
TOP Cherry	SUN437Q
PERIO PLUS	SUN438P



Characteristics

- Clinically proven formulations, approved as a medical device
- Combination product PERIO PLUS is registered for sub- and supragingival treatment with polishing effect ("two in one" - saving a final polish)
- Non-cariogenic formulations



General Information

Prophylaxis powder for professional teeth cleaning and biofilm (plaque) removal. All powders are very fine and evenly grained. Thus, one achieves a selective abrasion of coverings and works at the same time gently on soft tissue and dental hard tissues.

→ Professional powders for sub- and supragingival treatment available in many different flavors.

Application areas / indications

- TOP
Classic supragingival applications for the removal of biofilm (plaque) and severe discoloration. If necessary, prepare the tooth surface for various treatments.
- PERIO PLUS (patent pending)
The most versatile, time-saving, non-cariogenic and pleasant-tasting powder with no need for subsequent polishing. **Sub- and supragingival** (indications above and below the gingival margin) can be applied to the natural tooth and implants.

MP Class I 

Composition

Prophylaxis Powder TOP: Sodium hydrogen carbonate base

Prophylaxepulver PERIO PLUS: Tagatose Basis

Chemical-Physical Data

Optics: white, fine and homogeneous powders of defined grain size:

TOP:

median grain size 40µm, 300g


PERIO PLUS:

median grain size 15µm, 120g

Most versatile and efficient treatment!

Additional indications:

- Cleaning of implant surfaces
- Gentle cleaning in orthodontics (e.g. brackets)
- Cleaning of teeth before bleaching
- Cleaning of the tongue and fissure, as well as preventing halitosis

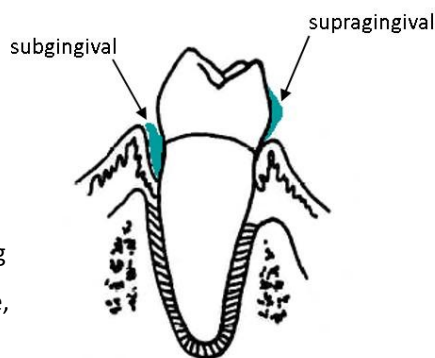

Variations

Available in the following flavors

TOP: Mint, Lemon, Cherry

PERIO PLUS: neutral

Instructions for use

Depending on the indication, use per polishing unit manufacturer's instructions. Store powder in a dust-free and dry place and do not mix with other powders.

TOP: Observe the polishing unit manufacturer's instructions for use and maximum filling height of the device. Not suitable for the removal of coverings or calculus in the periodontal pocket, do not aim the powder at the gums or other soft tissue. Exposed dentin and enamel caries must not be irradiated, since higher abrasives are to be expected here. Composite, compomer, amalgam and gold fillings should also not be exposed, as roughening may occur. Patients suffering from chronic bronchitis or asthma should never be treated with an air polishing unit. Patients on a low-sodium diet (such as high blood pressure) should not be treated with the powder because it contains sodium bicarbonate. Patients who are suspected of being allergic to flavoring agents should not be treated with the powder. Patient and operator must wear eye protection. The operator must wear a mask and gloves. Use high performance suction device. Apply Vaseline to the patient's lips. Only for dental use. Keep container closed. Shake well before use!

Report

Our products are scientifically tested and tested according to the latest methods. We are happy to send you copies of our tests and test reports on request. (e.g., grain size distribution measurements)

Delivery Forms

PROPHY TOP:

300g Bottle white with adapter closure

PROPHY PERIO PLUS:

120g Bottle white with adapter closure

Contact

PRISMAN Pharma International AG
Am Stalden 16
4622 Egerkingen
Switzerland

Tel.: +41 (0)62 206 20 60

Fax: +41 (0)62 206 20 69

www.innocid.com
info@prismanpharma.com

Manufacturer

PRISMAN GmbH
Otto-Hahn-Ring 6-18
64653 Lorsch
Germany

PERIO PLUS: Please note the instructions for use and the maximum filling height of the polishing device.

- Subgingival application:

Removal of periodontal biofilms.

- Additional supragingival application at PERIO PLUS:

Removal of supragingival biofilms and discoloration around the sulcus.

Patients suffering from chronic bronchitis or asthma should never be treated with a powder blasting device. The patient and operator must wear eye protection. The operator must wear a mask and gloves. Use high performance suction device. Apply Vaseline to the patient's lips. Only for dental use. Keep container closed. Shake well before use!